

F&P Eson™ 2

NASAL MASK



Confidence comes easy with Eson 2

F&P Eson 2 was designed to meet the needs of patients and sleep professionals, at every important milestone in the CPAP therapy journey.

OVER 20 DESIGN
IMPROVEMENTS
COMPARED TO
F&P ESON!



Confidence comes easy... at every CPAP therapy milestone

To improve on a mask designed for performance and comfort, our development team focused on ease of use, providing you and your patient confidence at every step from titration through to adherence.

CPAP therapy journey

FOR THE SLEEP PROFESSIONAL



1

SLEEP TEST

Selecting a nasal mask for titration

F&P Eson 2 retains the same RollFit™ Seal technology as F&P Eson, designed to deliver an effective seal to lessen the likelihood of leaks.



2

FIRST PATIENT CONTACT

Selecting a patient's mask for home use

The Intuitive headgear provides fitting options for patients, including the ability to stretch the mask directly over the head.



3

SET UP

Fitting and demonstrating the mask

Blue highlights – VisiBlue™ – incorporated into the mask provide visual cues for simpler, quicker and more intuitive mask education.



4

USE

Patient begins using mask

A design focus on ease of fitting and ease of use lessens the likelihood of call-backs and improves patient confidence.



5

ADHERENCE


Long-term adherence

Eson 2 has been designed to not just work on the first night, but to increase the chance of long-term therapy success.



Eson™ 2

Key features and benefits

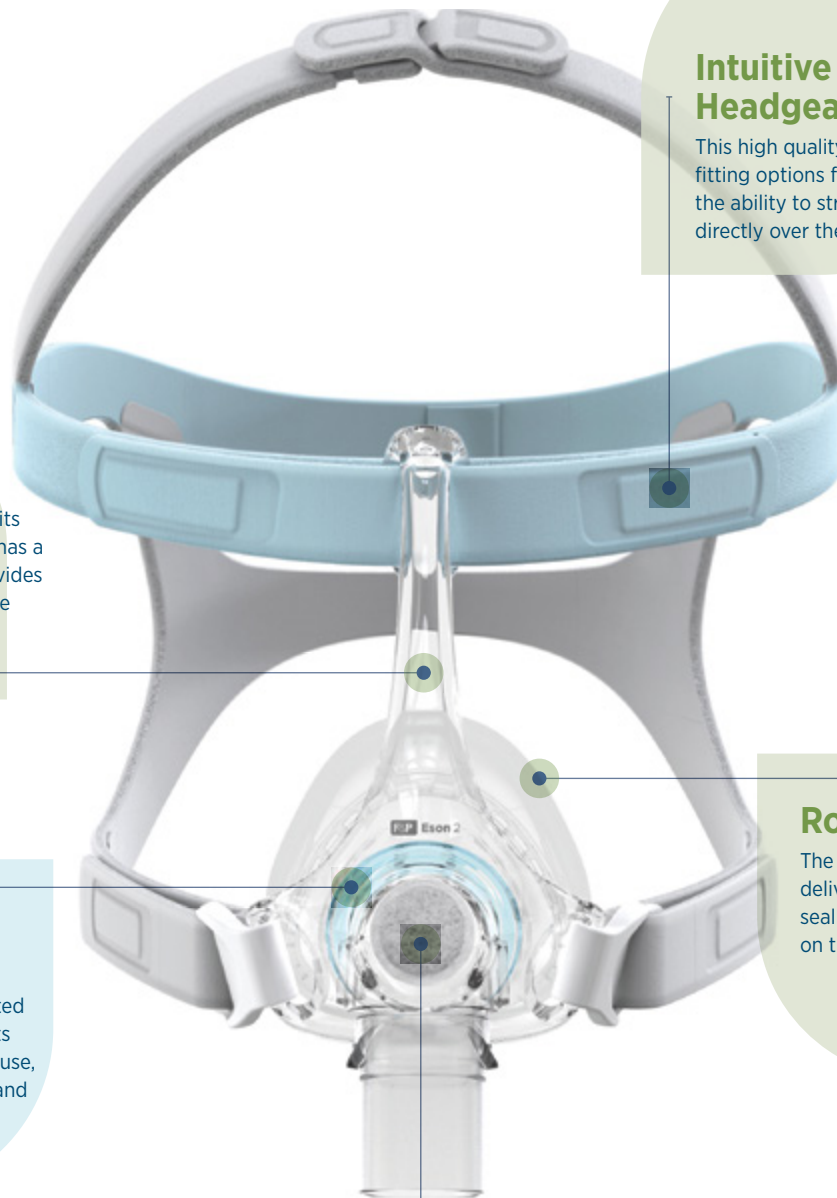

Intuitive Headgear

This high quality headgear provides fitting options for patients, including the ability to stretch the mask directly over the head.

Easy Frame

The Easy Frame - which fits all three seal sizes - now has a narrower profile. This provides a clearer line of sight while retaining seal stability.



VisiBlue™

Blue highlights incorporated into key mask components assist patients with mask use, orientation, disassembly and reassembly.

RollFit Seal

The RollFit Seal is designed to deliver a precise fit and effective seal by automatically adjusting on the bridge of the nose.

Washable Diffuser

The innovative diffuser, which reduces noise and draft, is now washable and more durable.

Fitting the Eson 2



Hold open the bottom straps of the Headgear with the Mask Frame facing downwards.



Guide the headgear over your head so that the silicone seal is resting on your forehead.



Place your hands on the bottom straps on either side of the mask frame and pull out and downwards. Ensure the silicone seal rests over your nose and the blue top strap is sitting on your forehead.



Gently tighten the headgear straps of the mask headgear starting with the blue forehead straps followed by the bottom straps.

NORTH AMERICA/ASIA-PACIFIC REORDER CODES

MASKS

ESN2SA	ESON2 Small	400ESN221	ESON2 Headgear Small
ESN2MA	ESON2 Medium	400ESN222	ESON2 Headgear Medium-Large
ESN2LA	ESON2 Large	400ESN231	ESON2 Mask No H/G Small
		400ESN232	ESON2 Mask No H/G Medium
		400ESN233	ESON2 Mask No H/G Large
		400FPH241	F&P Swivel 2
		400ESN251	ESON2 Clips
		400ESN261	ESON2 Diffuser
		400ESN271	ESON2 Elbow

SPARE PARTS

400ESN211	ESON2 Seal Small
400ESN212	ESON2 Seal Medium
400ESN213	ESON2 Seal Large

REST OF WORLD REORDER CODES

MASKS

ESN2SU	ESON2 Small	400ESN221	ESON2 Headgear Small
ESN2MU	ESON2 Medium	400ESN222	ESON2 Headgear Medium-Large
ESN2LU	ESON2 Large	400FPH241	F&P Swivel 2
		400ESN251	ESON2 Clips
		400ESN261	ESON2 Diffuser
		400ESN271	ESON2 Elbow

SPARE PARTS

400ESN211	ESON2 Seal Small
400ESN212	ESON2 Seal Medium
400ESN213	ESON2 Seal Large